

HOVEROUND®

Personal Mobility Solutions

MPV5®

Power Wheelchair



ATTENTION:

Please read the content
of your owner's manual
before operating your
power wheelchair.

OWNER'S MANUAL AND WARRANTY

6/14 D820087530

Appendix C: Locking Feature

Your MPV5 is equipped with a locking feature. Please refer to the appropriate instructions for the controller on your MPV5.

Lock Sequence System



Figure 9A - VSI Controller and new VSI Controller

1. Turn the controller on.
2. To lock the chair, depress and hold the ON/OFF button. After one second the controller will beep. Release the ON/OFF button. Move the joystick forward until the controller beeps. Move the joystick in reverse until the controller beeps. Release the joystick. There will be a long beep that indicates the controller is locked.
3. To unlock, turn the controller on. The speed/response indicator will be rippling up and down. Move the joystick forward until the controller beeps. Then move the joystick in reverse until the controller beeps. Release the joystick. There will be a long beep that indicates the controller is unlocked.



Phone: 1-800-96-HOVER (1-800-964-6837)
Fax: 1-800-747-9252

6010 Cattleridge Drive | Sarasota, Florida 34232-6061
www.hoveround.com