AEROLUX 2

COMFORT

by Travel Buggy

Ultra Lite Power Wheelchair
Carbon Fiber

TABLE OF CONTENTS

NOTICES	3	LEFT JOYSTICK INSTALLATION	15
SAFETY INFORMATION	4	FOLDING	16
BATTERY SAFETY INFORMATION	5	TRANSPORTING	17
SPECIFICATIONS	7	BATTERY MAINTENANCE	18
UNBOXING	9	CHARGING	20
SETUP	10	MAINTENANCE,CARE, TROUBLESHOOTING	22
CONTROLLER	13	DIAGNOSTIC CODES	23
FREE WHEEL MODE	14	LIMITED WARRANTY	24
SEAT WIDTH SPACERS	14	CONTACT & SUPPORT	25

 **CAUTION:** Read all precautions and instructions in this manual before using this equipment. Improper use or maintenance can void the warranty. Keep this manual for future reference.

NOTICES

Thank you for choosing AEROLUX 2 Comfort by Travel Buggy, a convenient to use and foldable power chair.

It is important to read through this manual before operating your AEROLUX 2C to become familiar with its features as well as its limitations and safety information

Operating and maintaining your AEROLUX 2C is designed to be straightforward and hassle-free. However, if you have any queries or need assistance, feel free to reach out to us at Travel Buggy.

OUR CUSTOMER HOTLINE

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

EMAIL

contact@travelbuggy.com

DISCLAIMER:

The notes, warnings, and cautions contained within the manual and marked by the triangular Caution Symbol  should be given special care. Users should also pay special attention to information marked in this manual beginning with **NOTICE**.

Travel Buggy, its affiliates and partners cannot be held responsible for personal injury or property damage resulting from the unsafe or improper operation or maintenance of the AEROLUX 2 Comfort Power Wheelchair.

As part of our ongoing product improvement initiative, Travel Buggy reserves the right to change specifications and design without notice. As a result, there may be minor differences between your AEROLUX 2C and the photos, illustrations and instructions in this manual.

SAFETY INFORMATION

- DO make certain that the seat is in a locked position when you are operating your AEROLUX 2C
- DO reduce your AEROLUX 2C's speed when you are driving around a corner
- DO proceed with caution as you approach the downgrade of a ramp or other incline and when driving near a raised surface or unprotected ledges and drop offs (curbs, porches, stairs, etc.)
- DO avoid exposure to rain, snow, ice, salt, or standing water whenever possible
- DO maintain and store in a clean and dry condition
- DO keep your hands away from the tires when driving. Be aware that loose fitting clothing can become caught in drive tires
- DO keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock
- DO follow the recommended weight capacity (300 lbs / 136 kg)
- DO follow the recommended incline slope rating (9 degrees)
- DO check the battery charge to ensure there is enough charge to get to your destination before using.
- DO ensure your AEROLUX 2C's power is OFF when entering and exiting the power wheelchair
- DO NOT drive your AEROLUX 2C across the side of a hill or diagonally up or down a hill, and do not stop (if possible) while driving up or down an incline
- DO NOT operate your new AEROLUX 2C for the first time without completely reading and understanding the owner's manual
- DO NOT carry additional passengers on your AEROLUX 2C
- DO NOT use your AEROLUX 2C as a seat in a moving vehicle
- DO NOT allow unsupervised children to play near the power wheelchair
- DO NOT operate your AEROLUX 2C while under the influence of alcohol, as this may impair your ability to drive safely
- DO NOT use the AEROLUX 2C as a seat in a moving vehicle. In the event of an accident or sudden stop, there is a significant risk of injury or damage, as you may be thrown from the power wheelchair
- ALL servicing must be performed by qualified personnel only. Unauthorized modifications to power wheelchair, its power system or its control system may compromise the performance and safety of the system. UNDER NO CIRCUMSTANCES should you attempt any repair or service that is not covered in this manual.
- ALWAYS ensure all safety warning labels are in place and legible.

WARNING: The operator should read all of the warning signs and instructions here and on the power wheelchair before use.

WARNING: You should not operate your power chair on public streets and roadways. Be aware that it may be difficult for traffic to see you when you are seated on your power wheelchair. Obey all local pedestrian traffic rules. Wait until your path is clear of traffic, and then proceed with extreme caution.

BATTERY SAFETY INFORMATION

WHEN USING THE BATTERY

1. Lithium-ion battery packs may get hot, explode or ignite and cause serious injury if exposed to abuse conditions. Be sure to follow the safety warnings listed below:
 - DO NOT place the battery near fire or heat the battery
 - DO NOT install the battery backwards so the polarity is reversed
 - DO NOT connect the positive and negative terminal of the battery to each other with any metal object (such as a wire)
 - DO NOT carry or store the battery together with necklaces, hairpins, or other metal objects
 - DO NOT pierce the battery with nails strike the battery with a hammer, step on the battery or otherwise subject it to strong impacts or shocks
 - DO NOT solder directly onto the battery
 - DO NOT expose battery to water or salt water, or allow the battery to get wet
2. DO NOT disassemble or modify the battery. The battery contains safety and protection devices, which if damaged, may cause the battery to generate heat, explode or ignite
3. DO NOT place the battery in or near fire, on stoves or other high temperature locations. Do not place the battery in direct sunlight, or use or store battery inside cars in hot weather. Doing so may cause the battery to generate heat, explode or ignite. Using the battery in this manner may also result in a loss of performance and a shortened life expectancy.
4. If the device is to be used by children, the caregiver should explain the contents of this document to the children and provide adequate supervision to ensure the device is being used appropriately
5. When the battery is worn out, insulate the terminals with adhesive tape or similar materials before disposal
6. Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact the Fire Department if any of these problems are observed
7. DO NOT place battery in microwave oven, high pressure containers or on induction cookware

BATTERY SAFETY INFORMATION

CHARGING PRECAUTIONS

1. BE SURE to follow the warnings listed below while charging the battery. Failure to do so may cause the battery to become hot, explode or ignite and cause serious injury
 - When charging the battery, ONLY USE THE SPECIFIED CHARGER PROVIDED WITH THE PRODUCT
 - DO NOT attach the battery to a power supply plug or directly to a car's cigarette lighter
 - DO NOT place the battery in or near fire, or in direct sunlight. Heating the battery can damage the safety circuitry, which can cause additional heating, rupture or ignition of the battery
2. DO NOT continue charging the battery if it does not recharge in the timelines specified in the operating manual
3. THE TEMPERATURE range over which a battery can be charged is 5°C to 40°C (41°F to 104°F). Charging the battery at temperatures outside of that range may cause severe damage to the battery or reduce battery life expectancy
 - Always protect battery from freezing temperatures and never charge a frozen battery. This will damage the battery and may cause personal injury. Attempting to charge a battery in freezing conditions does not prevent a battery from freezing

WHEN DISCHARGING THE BATTERY

1. DO NOT use the battery with any device other than the AEROLUX 2C. Utilizing the battery in devices other than the AEROLUX 2C can lead to damage or a reduction in its lifespan. Additionally, abnormal current flow caused by other devices may result in the battery overheating, potentially leading to explosion, ignition, and serious injury
2. THE TEMPERATURE range over which the battery can be discharged is -20°C to 60°C (-4°F to 140°F). Use of the battery outside this temperature range may damage the performance of the battery or may reduce its life expectancy

SPECIFICATIONS

SPECIFICATIONS

PERFORMANCE SPECIFICATIONS

DRIVING RANGE: 9.3 MI / 15 KM

MAXIMUM SPEED: 3.7 MPH / 6 KPH

DRIVE SYSTEM: TWO MOTOR DRIVE SYSTEM

MOTOR TYPE: 200 WATT BRUSHLESS

INCLINE CAPABILITY: 9° / 15.9%

TURNING RADIUS: 42.5"

STOPPING DISTANCE: 4.5 FT (AT FULL SPEED)

OBSTACLE CLEARANCE: 1.5"

BRAKING SYSTEM: ELECTROMAGNETIC

MAX WEIGHT CAPACITY: 300 LBS

DIMENSIONAL SPECIFICATIONS

UNFOLDED DIMENSIONS:
38.5"L X 24"W X 34"H

CHAIR GROSS WEIGHT: 37.5 LBS

CHAIR NET WEIGHT: 33 LBS
(EXCLUDING BATTERY AND SEAT CUSHION)

SEAT CUSHION SIZE: 18"(W) X 16"(D)

FOLDED DIMENSIONS:
31"L X 12"W X 29"H

SEATING WIDTH: W/O SPACERS 19"
W/ SPACERS 20"

FRONT WHEEL DIAMETER: 8"

REAR WHEEL DIAMETER: 12"

FLOOR TO SEAT HEIGHT: 21"

BACK SUPPORT HEIGHT: 16"

OTHER SPECIFICATIONS

BATTERY TYPE: LITHIUM ION
24V/10.4AH

BATTERY PROVIDED: ONE (1)

BATTERY WATT HOUR: 250 WH
(AIRLINE APPROVED)

BATTERY CHARGE CYCLES: ~1000

TIRES: NON-PUNCTURE

CHARGING TIME: 3.5 HOURS

CHARGER INPUT: 100-240V AC,
50-60HZ 2.7/1.8A (INTERNATIONAL)

CHARGER OUTPUT: 24V DC/3A

LAP BELT: PROVIDED

HORN: BUILT-IN

UNBOXING

TOP

BOTTOM

- To remove the power chair, orientate the box so that when it is laid down on the ground the arm rests are horizontally parallel to the ground
- Grab the arm rests and begin to pull your Travel Buggy out

- Locate the components provided with your power wheelchair

JOYSTICK CONTROLLER

LITHIUM-ION BATTERY

BATTERY CHARGER

FOOT RESTS X2

USB TYPE C CONVERTER

TOOL KIT

SEAT WIDTH SPACER KIT

CUP HOLDER

LAP BELT

SEAT CUSHION

BACKREST CUSHION

LEG SUPPORT

 NOTICE: Lithium battery is installed in the equipment when shipped.

SETUP

UNFOLDING

- With the AEROLUX 2C standing upright, pull both armrests in opposite directions to start the unfolding process
- Press down on the seat until you hear the seat lock in place

WARNING: When unfolding the seat, please keep your hand flat on the surface. This is to prevent your fingers from being pinched. Avoid wrapping your fingers around the seat frame (image 3).

- Swivel up the push handles 90° until it locks into place

INSTALL CUSHIONS

- Place the back rest cushion on the back frame and fold the top flap to secure the velcro behind the chair (image 1)
- Place the seat cushion on the seat frame to secure the velcro

NOTICE: Failure to fully extend the power chair into its locked position may result in injury while driving.

SETUP


- Slide the joystick controller into the armrest tube until a comfortable arm length is achieved for the user (image 1).
- Tighten the corkscrew to secure the joystick (image 2)
- Align the arrows of the joystick connection cables and press them together to plug the joystick controller in (image 3)
- Rotate the coupling over the connection point to secure the connection point

WARNING: Align the joystick connector's pins with the holes carefully prior to pressing them together. Improper alignment or forceful pressing can bend or break the pins, making the joystick inoperable.

INSTALL FOOT RESTS & LEG SUPPORT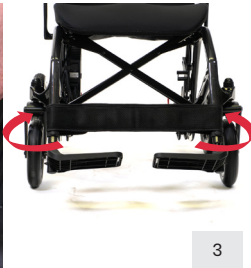

- The AEROLUX 2C is shipped with the foot rests not installed. The foot rests need to be inserted prior to driving
- Slide the foot rests into the tube located on the front castor with the holes aligned (image 1)
- Once the foot rest is inserted all the way and the holes on the tube and the foot rest are aligned, insert the locking pin in through the holes until the clamp is wrapped around the tube (image 2)
- OPTIONAL: Wrap the leg support strap around the front forks and secure the velcro behind the posts (image 3)

WARNING: Failure to align the joystick connector pins properly before insertion may cause the pins to bend or break, making the joystick inoperable. Damage caused due to improper installation may not be covered under warranty.

SETUP


- Before sitting in the Power Chair, be sure the power button is OFF and the rear brake levers are in the DOWN (locked) position
- AEROLUX 2C offers the user 2 optional entry alternatives, direct front entry or side entry
- If you prefer a front entry, we recommend flipping up the foot plate to create an easier sitting experience
- DO NOT step on the footplate when entering or exiting the AEROLUX 2C as it may cause the power chair to move in an unsafe manner

- To enter from the side of the chair, press the arm rest release button (image 1). Raise the arm rest back and away. When seated, lower arm rest back into position and ensure it is locked

CHECK IF THE BATTERY IS PROPERLY INSERTED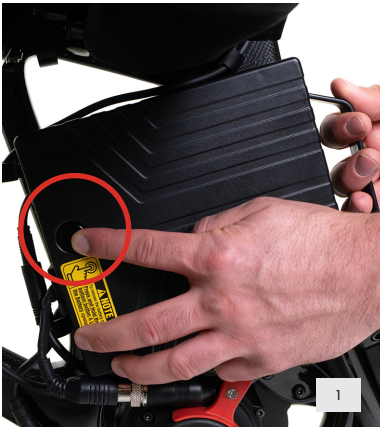

- Locate the battery on the side of the chair, the quick release button (image 1) will be popped out indicating that the battery is properly secured into place

⚠ NOTICE: Make sure the battery is connected to the system or else the chair will not work

CONTROLLER

7-Speed Control Lights

SPEED CONTROL LIMITS

- 1-Light On 2.8 KPH / 1.7 MPH
- 2-Lights On 3.3 KPH / 2 MPH
- 3-Lights On 3.8 KPH / 2.4 MPH
- 4-Lights On 4.3 KPH / 2.7 MPH
- 5-Lights On 4.8 KPH / 3 MPH
- 6-Lights On 5.3 KPH / 3.3 MPH
- All 7-Lights On Max Speed

Power ON/OFF

POWER ON/OFF

Press the RED button to turn the power chair ON or OFF.

Mute

Press the  button to mute the joystick operation. Press again to resume the sound operations.

Speed Control

INCREASE SPEED

Press the **PLUS +** button to increase the speed

DECREASE SPEED

Press the **MINUS -** button to decrease the speed

Battery Indicator Lights

There are a total of 5 battery power indicator lights. From top down the lights will disappear as the battery drains indicating the amount of charge remaining. Recharge at any point before the last light disappears, indicating a fully depleted battery.

Horn

Press the  button to sound the horn.

Light

Press the  button to turn on the light. Press again to dim the light. Press three times to turn on strobe effect. Press four times to turn off light.

- AEROLUX 2C Power Chair has an easy to use 360 degree joystick controller featuring multiple speeds and a battery life indicator
- Take some time in a safe, controlled environment to learn the operation of the controller, the speed and the maneuverability of the AEROLUX 2C
- Be sure to adjust the speed depending on your environment and while turning
- To slow down, slowly bring the joystick back toward the initial position.
- To stop, just bring the joystick back to the initial position
- Use extra care when you drive the AEROLUX 2C in reverse

FREE WHEEL MODE


To allow a caregiver to push the AEROLUX 2C manually, it must be placed in Freewheel Mode.

To activate Freewheel Mode:
Bring the chair to a complete stop and turn OFF the power.
Push UP on the two red levers located on each rear wheel, as shown.

To re-engage the chair for POWER mode, press the levers down.

SEAT WIDTH SPACERS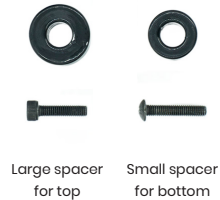

- To adjust the width of your Aerolux 2C, start by removing the armrests. Unscrew the bolts at the top and bottom of the mounting points (image 1).
- Place the small spacer at the bottom and the large spacer at the top. (image 2).
- Reattach the arm rests by securing the bolts. (image 3 & 4)

⚠ NOTICE: If the chair beeps and won't move, there is a good chance the motors remain in FREE WHEEL mode. Refer to the TROUBLE SHOOTING section for more details.

LEFT JOYSTICK INSTALLATION

1

2

- Turn off the power and **remove the battery from the chair.** (image 1)
- Disconnect the joystick cable from the CPU (the black box under the chair). (image 2)
- Remove the joystick from the right armrest and install it on the left armrest.

3

4

- Trace the joystick cable from the CPU and unclip it from any cable holders along the frame.
- Plug in the left-side joystick cable to the CPU and begin securing it along the same path using the existing clips.
- Run the cable along the back of the seat base, under the velcro strap holding the left motor cable. (image 3)
- Continue routing the cable up and under the left armrest, then connect it to the joystick. (image 4)
- Reinstall the battery and turn the chair back on.
- **Ensure that all controls are working properly before using the chair.**

FOLDING

- Press the push handle levers on either side to disengage and push them 90° downwards

WARNING: To ensure your safety when folding the AEROLUX 2C, please confirm that your feet are positioned outside of the wheelchair's wheels before proceeding. This precaution is vital to prevent the risk of your feet being pinched or injured by the wheels.

- Fold both footrests up 90°


- Pull the 2 handles just under the seat cushion upwards to start the folding process

- Press both armrests towards each other to compress the power chair into a fully folded position

TRANSPORTING

1

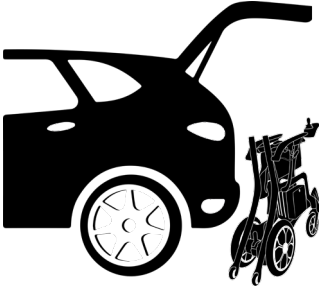2

3

4

- To load the AEROLUX 2C power chair into the back of a vehicle for transport we recommend utilizing the back of the vehicle as leverage
- Position the power chair against the car as indicated in Image 1 (Note: we recommend using the Travel Buggy Bumper Blanket to reduce scratching of the vehicle)
- Grasp the front and rear wheels (Image 2)
- Lift against the back of the vehicle, leveraging the power wheelchair into the trunk of the vehicle (Image 3)
- Push the AEROLUX 2C into the desired position within the vehicle (Image 4)

⚠ WARNING: Never transport the AEROLUX 2C in a tie down system, as the AEROLUX 2C is not compatible with them. Never sit in the AEROLUX 2C and be transported in a moving vehicle. Do not place the folded AEROLUX 2C in the front seat with the driver where it could move or slide.

BATTERY MAINTENANCE

Lithium-Ion rechargeable battery require routine maintenance and care in their use and handling. Read and follow the guidelines in this document to safely use Lithium-Ion battery and achieve the maximum battery life span.

OVERVIEW

Do not leave battery unused for extended periods of time, either in the product or in storage. When a battery has been unused for 6 months, check the charge status and charge or dispose of the battery as appropriate.

The typical estimated life of a Lithium-Ion battery is about three years or ~1000 charge cycles, whichever occurs first. One charge cycle is a period of use from fully charged, to fully discharged, and fully recharged again. Use a three-year life expectancy for battery that do not run through complete charge cycles.

Rechargeable Lithium-Ion battery have a limited life and will gradually lose their capacity to hold a charge. This loss of capacity (aging) is irreversible. As the battery loses capacity, the length of time it will power the product (run time) decreases.

Lithium-Ion battery continue to slowly discharge (self-discharge) when not in use or while in storage. Routinely check the battery's charge status.

Use only the provided battery(s) in your product.

BATTERY MAINTENANCE

Observe and note the run time that a new fully-charged battery provides for powering your product. Use this new battery run time as a basis to compare run times for older battery. The run time of your battery will vary depending on the product's configuration and the applications that you run.

Routinely check the battery's charge status.

Carefully monitor battery that are approaching the end of their estimated life.

Consider replacing the battery with a new one if you note either of the following conditions:

- The battery run time drops below about 80% of the original run time.
- The battery charge time increases significantly.

If a battery is stored or otherwise unused for an extended period, be sure to follow the storage instructions in this document. If you do not follow the instructions, and the battery has no charge remaining when you check it, consider it to be damaged. Do not attempt to recharge it or to use it. Replace it with a new battery.

CHARGING

Always adhere to the charging instructions for your product, which can be found on page 6 of this manual. This section provides detailed information about charging the battery, ensuring proper use and care.

Use only the provided charger to charge your battery(s).

STORAGE

BATTERY MAINTENANCE

- Charge or discharge the battery to approximately 50% of capacity before storage.
- Charge the battery to approximately 50% of capacity at least once every six months.
- Remove the battery and store it separately from the product.
- Store the battery at temperatures between 5 °C and 20 °C (41 °F and 68 °F).

HANDLING PRECAUTIONS

- Do not disassemble, crush, or puncture a battery.
- Do not short the external contacts on a battery.
- Do not expose a battery to fire or water.
- Do not expose a battery to temperatures above 60 °C (140 °F).
- Keep the battery away from children.
- Avoid exposing the battery to excessive shock or vibration.
- Do not use a damaged battery.
- If a battery pack has leaking fluids, do not touch any fluids.
- Dispose of a leaking battery pack (see Disposal and Recycling in this document).
- In case of eye contact with fluid, do not rub eyes. Immediately flush eyes thoroughly with water for at least 15 minutes, lifting upper and lower lids, until no evidence of the fluid remains. Seek medical attention.
- Immediately discontinue use of the battery if, while using, charging or

storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact the Fire Department if any of these problems are observed

TRANSPORTATION

- Always check all applicable local, national, and international regulations before transporting a Lithium-Ion battery.
- Transporting an end-of-life, damaged, or recalled battery may, in certain cases, be specifically limited or prohibited.

DISPOSAL AND RECYCLING

- Lithium-Ion battery are subject to disposal and recycling regulations that vary by country and region. Always check and follow your applicable regulations before disposing of any battery. Contact Rechargeable Battery Recycling Corporation (www.rbcc.org) for U.S.A. and Canada, or your local battery recycling organization.
- Many countries prohibit the disposal of waste electronic equipment in standard waste receptacles.
- Place only discharged battery in a battery collection container. Use electrical tape or other approved covering over the battery connection points to prevent short circuits.

 **NOTE:** The battery self-discharges during storage. Higher temperatures (above 20 °C or 68 °F) reduce the battery storage life.

CHARGING

Onboard Charging

Offboard Charging

- Upon unboxing and set up, first check the charge of the battery before immediately assuming there is a requirement to re-charge the battery
- Avoid unnecessary re-charging to maintain the life of your battery

ONBOARD CHARGING (PREFERRED METHOD)


- Turn OFF the power to the AEROLUX 2C
- Connect the charger (XLR male) to the controller port (XLR female)
- Connect the input plug of the charger with your local power source
- Check on the charger brick that the light is red (charging)

OFFBOARD CHARGING

- Turn OFF the power to the AEROLUX 2C
- To remove the battery, first press the battery release button (image 1) while simultaneously sliding the battery upwards and out
- **IMPORTANT!** Make sure that your fingers are not placed between the battery and the chair frame to avoid getting injured.
- (Image 2) Connect the charger (XLR male) to the battery charging port (XLR female)
- Connect the input plug of charger with your local power source
- Please ensure the charger and battery are on a flat surface and keep in a well ventilated area
- Once charging is complete, remove the power cord and ensure that the battery is re-inserted correctly back into the power chair prior to driving.

⚠ WARNING: TO REDUCE THE RISK OF INJURY, CHARGE LITHIUM-ION PROVIDED BATTERY ONLY WITH PROVIDED LITHIUM-ION BATTERY CHARGER. Other types of chargers may cause the battery to generate heat, explode or ignite potentially causing personal injury or damage.

CHARGING

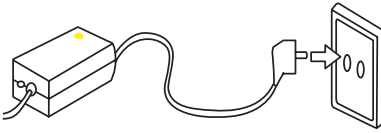- Charging
- Fully Charged

LED Battery Charge

80%-100%

60% - 80%

40% - 60%

20% - 40%

0% - 20%

- When charging the indicator light on the power charger brick is yellow
- After 4 to 6 hours of charging (depending on the remaining battery power quantity), the indicator light changes from yellow to green, which means the battery is fully charged
- To check your battery charge, unplug the charger from the joystick and turn ON the joystick controller power
- There are a total of 5 battery power indicator lights
- From top to down the lights will begin to disappear as the battery(s) drain indicating the amount of charge remaining
- Recharge at any point before the last light disappears indicating a fully depleted battery(s)

**BATTERY
CHARGE
INDICATOR
ON JOYSTICK
CONTROLLER**

⚠ WARNING: The battery charger brick may get very hot (up to 60 degrees) this is normal operation, please be careful when handling the battery charger.

⚠ WARNING: Check the battery charge indicator prior to driving the power chair to ensure there is enough charge to reach your destination.

MAINTENANCE, CARE, TROUBLESHOOTING

It is important to perform regular maintenance of your AEROLUX 2C for longevity and warranty eligibility. Even a little cleaning now and again will go a long way to keeping your device looking and functioning like new.

Certified repair shops with trained professionals who specialize in Power Chairs and scooters will be qualified to perform the service and maintenance on your AEROLUX 2C Power Chair should it be required. Please be sure to keep a log of the service and maintenance performed. Your shop or technician may require parts, which Travel Buggy has available.

We recommend you inspect your AEROLUX 2C two to three times per year for in the following areas:

- **Frame:** tighten loose screws as necessary using the tool kit included with your AEROLUX 2C.
- **Wheels:** make sure they rotate easily. Significant resistance may require new bearings.
- **Armrests:** look for excessive wear or if they are loose.

Replacement parts and more major maintenance issues should be done by a professional. We do not recommend that you do intense types of maintenance work to the AEROLUX 2C yourself.

LUBRICATION

- If a persistent squeaking noise begins to occur follow these steps:
 1. Locate the area of the squeak, usually the front wheel (bearing) or front fork areas.
 2. Remove the parts and clean any debris from the area
 3. Lubricate the fitting components with a wheelchair or bike lubricant prior to putting it back together
 4. Do not over-tighten the bolts when you put it back together. If the squeaking continues or gets worse contact a Travel Buggy service representative
- Your AEROLUX 2C power wheelchair has electric hub motors that do not require lubrication, please do not lubricate the motors.

Question	Possible Causes	Answer
Why is there NO POWER to the AEROLUX 2C?	<ul style="list-style-type: none">• Battery is not inserted correctly• Battery power is too low.• Joystick is not connected properly	<ul style="list-style-type: none">• Remove and re-insert the battery, make sure it is fully inserted into its mounting port on the side of the chair.• Charge the Battery• Disconnect and reconnect the joystick
Why does my power chair beep and not move?	<ul style="list-style-type: none">• Freewheel mode engaged (Code C4)• Other error code	<ul style="list-style-type: none">• Ensure the rear motor switches are in POWER mode, rear wheels should be locked• Contact Travel Buggy if other error code present

⚠ WARNING: DO NOT charge the battery for greater than 8 hours, this will significantly reduce the longevity of the lithium ion battery and be of harm to the user.

DIAGNOSTIC CODES

Diagnostic Code	Instruction	Solution
1	Low Power	The battery has low voltage and needs to be recharged.
2.3	Left Motor Malfunction	Check left motor connection and whether it was in freewheel when turned on.
4.5	Right Motor Malfunction	Check right motor connection and whether it was in freewheel when turned on.
6	Over Current	Whether the motor drive mechanism is stuck, the current detected by the ammeter is not big, it may be an electrical fault in the controller.
7	Joystick	The joystick knob was not centered when powered on, or the connection may be loose.
8	Joystick Malfunction	Please consult your dealer
9	Communication Error	Please consult your dealer

LIMITED WARRANTY

The purchaser of this product is entitled to a limited warranty as offered by Travel Buggy and its affiliates on the following components and timeframes from the date of purchase:

5 YEAR WARRANTY - Main Frame

2 YEAR WARRANTY - Motor

1 YEAR WARRANTY - Control System (CPU and Controller), Battery

90 DAY LIMITED WARRANTY - On the wearing parts, such as the tires, armrest, joystick button covers, controls knobs, seat cushion and backrest padding.

This warranty is valid for the replacement of dysfunctional parts only. Any parts under warranty will be replaced and shipped for free. Any service and labor fees, if applicable, to replace parts under warranty must be paid by the user.

Due to its straightforward design, most parts can be easily exchanged by the end user without a professional service tech required. However, it is always recommended you seek professional help for maintenance and service, to make sure the work is done properly.

The warranty only covers NON-DELIBERATE

Damage.

The warranty does not cover:

- Products damaged by user negligence.
- Products damaged accidentally.
- Products damaged intentionally.
- Products that have been subjected to negligence.
- Products that have been subjected to abuse.
- Products that have been improperly stored.
- Products that have been improperly handled.
- Products that have been improperly operated.
- Products that have experienced general misuse.
- Products that have been modified in an unauthorized, unapproved way.

Warranty is non-transferable and only valid for the original wheelchair purchaser.

Travel Buggy will, at its sole discretion, replace any warrantable item that fails within the warranty period.

SERIAL NUMBER

The serial number on the AEROLUX 2C is located on the inside of the horizontal frame bar just above the front left wheel. It is important to mention the serial number in any warranty related communications

CONTACT & SUPPORT

Operating and maintaining your AEROLUX 2C is designed to be straightforward and hassle-free. However, if you have any queries or need assistance, feel free to reach out to us at Travel Buggy.

OUR CUSTOMER HOTLINE

1.888.56.BUGGY (Canada)

1.855.56.BUGGY (USA)

EMAIL

contact@travelbuggy.com

Additional images and videos are available on our website.

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