CITY 2 PLUS

by Travel Buggy

Foldable Power Chair

TABLE OF CONTENTS

NOTICES	3
SAFETY INFORMATION	4
BATTERY SAFETY INFORMATION	5
SPECIFICATIONS	7
UNBOXING	9
UNFOLDING	10
FOLDING	11
SET UP	12
CONTROLLER	14

HOW-TO FEATURES	
• SEATING ADJUSTMENT	15
• WIDTH ADJUSTMENT	15
• REMOVABLE MOTORS	16
• FREEWHEEL	16
TRANSPORTING	17
BATTERY MAINTENANCE	18
CHARGING	20
MAINTENANCE, CARE, TROUBLESHOOTING	22
LIMITED WARRANTY	23
CONTACT & SUPPORT	24

NOTICES

Thank you for choosing CITY 2 PLUS by Travel Buggy, a convenient to use and portable powerchair.

It is important to read through this manual before operating your CITY 2 PLUS to become familiar with its features as well as its limitations and safety information.

You will find the operation and care of your CITY 2 PLUS very simple and easy, but should you have any questions please contact Travel Buggy.

OUR CUSTOMER HOTLINE

Canada: 1.888.56.BUGGY (28449)

USA: 1.855.56.BUGGY (28449)

EMAIL

contact@travelbuggy.com

DISCLAIMER:

The notes, warnings, and cautions contained within the manual and marked by the triangular Caution Symbol  should be given special care. Users should also pay special attention to information marked in this manual beginning with **NOTICE**.

Travel Buggy, its affiliates and partners cannot be held responsible for personal injury or property damage resulting from the unsafe or improper operation or maintenance of the CITY 2 PLUS Power Wheelchair.

As part of our ongoing product improvement initiative, Travel Buggy reserves the right to change specifications and design without notice. As a result, there may be minor differences between your CITY 2 PLUS and the photos, illustrations and instructions in this manual.

SAFETY INFORMATION

- DO make certain that the seat is in a locked position when you are operating your CITY 2 PLUS
- DO reduce your CITY 2 PLUS's speed when you are driving around a corner
- DO proceed with caution as you approach the downgrade of a ramp or other incline and when driving near a raised surface or unprotected ledges and drop offs (curbs, porches, stairs, etc.)
- DO avoid exposure to rain, snow, ice, salt, or standing water whenever possible
- DO maintain and store in a clean and dry condition
- DO keep your hands away from the tires when driving. Be aware that loose fitting clothing can become caught in drive tires
- DO keep tools and other metal objects away from battery terminals. Contact with tools can cause electrical shock
- DO follow the recommended weight capacity (395 lbs / 180 kg)
- DO follow the recommended incline slope rating (12 degrees / 21.3% slope)
- DO check the battery charge to ensure there is enough charge to get to your destination before using.
- DO ensure your CITY 2 PLUS's power is OFF when entering and exiting the power wheelchair
- DO NOT drive your CITY 2 PLUS across the side of a hill or diagonally up or down a hill, and do not stop (if possible) while driving up or down an incline
- DO NOT operate your new CITY 2 PLUS for the first time without completely reading and understanding the owner's manual
- DO NOT carry additional passengers on your CITY 2 PLUS
- DO NOT use your CITY 2 PLUS as a seat in a moving vehicle
- DO NOT allow unsupervised children to play near the power wheelchair
- DO NOT operate your CITY 2 PLUS while under the influence of alcohol, as this may impair your ability to drive safely
- DO NOT sit in the CITY 2 PLUS while in a moving vehicle, an accident or sudden stop may cause you to be thrown from the power wheelchair
- ALL servicing must be performed by qualified personnel only. Unauthorized modifications to power wheelchair, its power system or its control system may compromise the performance and safety of the system. UNDER NO CIRCUMSTANCES should you attempt any repair or service that is not covered in this manual.
- ALWAYS ensure all safety warning labels are in place and legible.

 **WARNING:** The operator should read all of the warnings signs and instructions here and on the power wheelchair before use.

 **WARNING:** You should not operate your power wheelchair on public streets and roadways. Be aware that it may be difficult for traffic to see you when you are seated on your power wheelchair. Obey all local pedestrian traffic rules. Wait until your path is clear of traffic, and then proceed with extreme caution.

BATTERY SAFETY INFORMATION

WHEN USING THE BATTERY

1. Lithium-ion battery packs may get hot, explode or ignite and cause serious injury if exposed to abuse conditions. Be sure to follow the safety warnings listed below:
 - DO NOT place the battery near fire or heat the battery
 - DO NOT install the battery backwards so the polarity is reversed
 - DO NOT connect the positive and negative terminals of the battery to each other with any metal object (such as a wire)
 - DO NOT carry or store the battery together with necklaces, hairpins, or other metal objects
 - DO NOT pierce the battery with nails strike the battery with a hammer, step on the battery or otherwise subject it to strong impacts or shocks
 - DO NOT solder directly onto the battery
 - DO NOT expose battery to water or salt water, or allow the battery to get wet
2. DO NOT disassemble or modify the battery. The battery contains safety and protection devices, which if damaged, may cause the battery to generate heat, explode or ignite
3. DO NOT place the battery in or near fire, on stoves or other high temperature locations. Do not place the battery in direct sunlight, or use or store batteries inside cars in hot weather. Doing so may cause the battery to generate heat, explode or ignite. Using the battery in this manner may also result in a loss of performance and a shortened life expectancy.
4. If the device is to be used by children, the caregiver should explain the contents of this document to the children and provide adequate supervision to ensure the device is being used appropriately
5. When the battery is worn out, insulate the terminals with adhesive tape or similar materials before disposal
6. Immediately discontinue use of the battery if, while using, charging or storing the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed
7. DO NOT place battery in microwave oven, high pressure containers or on induction cookware

BATTERY SAFETY INFORMATION

CHARGING PRECAUTIONS

1. BE SURE to follow the warnings listed below while charging the battery. Failure to do so may cause the battery to become hot, explode or ignite and cause serious injury.
 - When charging the battery, ONLY USE THE SPECIFIED CHARGER PROVIDED WITH THE PRODUCT
 - DO NOT attach the battery to a power supply plug or directly to a car's cigarette lighter
 - DO NOT place the battery in or near fire, or in direct sunlight. Heating the battery can damage the safety circuitry, which can cause additional heating, rupture or ignition of the battery
2. DO NOT continue charging the battery if it does not recharge in the timelines specified in the operating manual
3. THE TEMPERATURE range over which a battery can be charged is 0°C to 45°C (32°F to 113°F). Charging the battery at temperatures outside of that range may cause severe damage to the battery or reduce battery life expectancy

WHEN DISCHARGING THE BATTERY

1. DO NOT discharge the battery using any other device except for the specified device. When the battery is used in devices other than the specified device, it may damage the battery or reduce its life expectancy. If the device causes abnormal current flow, it may cause the battery to become hot, explode or ignite and cause serious injury.
2. THE TEMPERATURE range over which the battery can be discharged is -20°C to 60°C (-4°F to 140°F). Use of the battery outside this temperature range may damage the performance of the battery or may reduce its life expectancy.

SPECIFICATIONS

SPECIFICATIONS

PERFORMANCE SPECIFICATIONS

- DRIVING RANGE: 15.5 MI / 25 KM
- MAXIMUM SPEED: 4.5 MPH / 7 KPH
- DRIVE SYSTEM: TWO MOTOR DRIVE SYSTEM
- MOTOR TYPE: 250 WATT BRUSHLESS
- INCLINE CAPABILITY: 12° / 21.3%
- TURNING RADIUS: 26"
- STOPPING DISTANCE: 5 FT (AT FULL SPEED)
- OBSTACLE CLEARANCE: 2"
- BRAKING SYSTEM: ELECTROMAGNETIC
- MAX WEIGHT CAPACITY: 395 LBS

DIMENSIONAL SPECIFICATIONS

- UNFOLDED DIMENSIONS: 44"L X 23"W X 40"H
- CHAIR GROSS WEIGHT: 65 LBS
- CHAIR NET WEIGHT: 58 LBS (NO BATTERIES)
- 3PCS SUBASSEMBLY WEIGHT: 40 LBS (FRAME), 12.5 LBS X 2 (MOTOR/WHEEL)
- SEAT CUSHION SIZE: 18.5" (W) X 18.5" (D)

- FOLDED DIMENSIONS: 33"L X 23"W X 13"H
- SEATING WIDTH: 19.5" / 23"
- FRONT WHEEL DIAMETER: 8"
- REAR WHEEL DIAMETER: 12"
- FLOOR TO SEAT HEIGHT: 19.5"
- BACK SUPPORT HEIGHT: 20.5"

OTHER SPECIFICATIONS

- BATTERY TYPE: LITHIUM-ION (24V X 6AH)
- BATTERIES PROVIDED: TWO (2)
- BATTERY WATT HOUR: 144 WH (AIRLINE APPROVED)
- BATTERY CHARGE CYCLES: ~1,000
- TIRES: NON-PUNCTURE
- CHARGING TIME: 5 TO 7 HOURS
- CHARGER INPUT: 100-240V, 50/60HZ (INTERNATIONAL)
- CHARGER OUTPUT: 29.4V 2.0A
- LAP BELT: PROVIDED
- HORN: BUILT-IN

UNBOXING

- To remove the powerchair, orientate the box such that when it is laid down on the ground the footplate is at the bottom
- Grab the top bar and begin to pull your Travel Buggy out horizontally parallel with the ground.

- Locate the components provided with your power wheelchair

SEAT CUSHION

LITHIUM
BATTERY (2)*

BACK REST
CUSHION

TOOLS

JOYSTICK
CONTROLLER

SEAT WIDTH
SPACERS X2

CUP HOLDER

BATTERY CHARGER

LAP BELT

BOLTS X4

 **NOTICE:** Lithium Batteries are already installed and charged on the power chair when shipped.

UNFOLDING

- Lay CITY 2 PLUS flat on the ground as illustrated above

- Hold the top bar and pull up the chair while pressing the edge of the seat to open similar to a lawn chair.

- Continue to unfold the power chair until it is in fully expanded seating position.

- Ensure the chair is fully extended and locked into the seating position.

 **NOTICE:** Failure to fully extend the power chair into its locked position may result in injury while driving.

FOLDING

- Press the red seat frame lock button down to release the chair for folding

- Continue the folding process by holding one hand beneath the seat frame and another on the top bar similar to a lawn chair

- Fold the footrest into the power chair before continuing the folding process

- The power chair is now completely folded. CITY 2 PLUS can also be folded on to its rear wheels and stands on its own for greater convenience.

SET UP

- Locate the joystick controller in the storage pouch below the power chair.
- Slide the joystick controller into the armrest tube until a comfortable arm length is achieved for the user (image 1).
- Tighten the corkscrew to secure the joystick
- Align the arrows of the joystick connection cables and press them together to plug the joystick controller in (image 2)
- Rotate the coupling over the connection point to secure the connection point

- Locate the cup holder provided with your new CITY 2 PLUS Power Chair
- Insert the cup holder into the opposite arm rest tube from joystick, fasten securely with the corkscrew

SET UP

- Before sitting in the electric wheelchair, be sure the power button is OFF and the rear brake levers are in the DOWN (locked) position
- CITY 2 PLUS offers the user 2 optional entry alternatives, direct front entry or side entry
- If you prefer a front entry, we recommend flipping up the foot plate to create an easier sitting experience
- DO NOT step on the footplate when entering or exiting the CITY 2 PLUS as it may cause the power chair to move in an unsafe manner

- To enter on the side, pull the arm release knob and lift the arm rest vertical to provide the required clearance
- To enter on the side, pull the arm release knob and lift the arm rest vertical to provide the required clearance

- Joystick can be positioned on left or right arm rest
- To change joystick position, first remove the joystick by performing the opposite sequence of the steps above
- Unscrew the wire harness under the armrest (1) cut the plastic ties on the frame of the chair (2) to create slack in the joystick power cable
- Move the power cable to the opposite side and re-fasten using the wire harness and plastic ties, be sure the cable is not interfering with the folding or movement of the electric wheelchair and connect the joystick

CONTROLLER

Speed Control

INCREASE SPEED

Press the **PLUS +** button to increase the speed

DECREASE SPEED

Press the **MINUS -** button to decrease the speed

Horn

Press the  button to sound the horn

5-Speed Control Lights

SPEED CONTROL LIMITS

-  1-Light On 1.5 MPH
-  2-Lights On 2.25 MPH
-  3-Lights On 3 MPH
-  4-Lights On 3.75 MPH
-  All 5-Lights On 4.5 MPH

Power ON/OFF

POWER ON

Press the **GREEN** button to turn ON the power chair

POWER OFF

Press the **RED** button to turn OFF the power chair

Battery Indicator Lights


There are a total of 7 battery power indicator lights: 3 Green, 3 Yellow and 1 Red. From top down the lights will disappear as the battery drains indicating the amount of charge remaining. Recharge at any point before the last Red light disappears, indicating a fully depleted battery.

- CITY 2 PLUS electric wheelchair has an easy to use 360 degree joystick controller featuring multiple speeds and a battery life indicator
- Take some time in a safe, controlled environment to learn the operation of the controller, the speed and the maneuverability of the CITY 2 PLUS
- Be sure to adjust the speed depending on your environment and while turning
- To slow down, slowly bring the joystick back toward the initial position.
- To stop, just bring the joystick back to the initial position
- Use extra care when you drive the CITY 2 PLUS in reverse

HOW-TO FEATURES

ADJUST WIDTH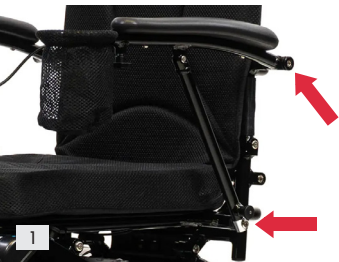


- Provided with your CITY 2 PLUS electric wheelchair are optional Seat Width Spacers to increase the seating area width an additional 3.5"
- To adjust the width of your Travel Buggy, first remove the arm rests by removing the bolts on the top and bottom of the arm rest (image 1)
- Locate the Seat Width Spacers and place the shorter cylinder at the bottom, longer cylinder at the top (image 2)
- Reattach the arm rests by securing the larger bolt on the bottom spacer and the smaller bolt on the top spacer (images 3 & 4). Repeat on each side

Video Tutorial Link:
www.vimeo.com/729278354


ADJUST SEATING

- The adjustable foot rest has 3 extension positions
- To adjust the foot rest to a comfortable length, pull both side pins to release the foot rest, then pull it forward to the subsequent position
- Ensure the pins are both locked into the new position
- Your CITY 2 PLUS model comes with a reclining backrest. This allows the user to recline the back rest into multiple positions for comfort
- To release the back rest, push UP on the lever at the back of the back rest (1) while simultaneously pulling down from the top bar of the back rest frame Release the lever to lock the back rest into the desired position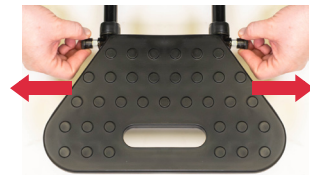


HOW-TO FEATURES

REMOVABLE WHEELS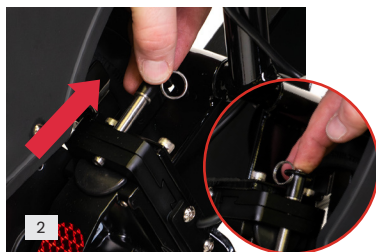

- Your CITY 2 PLUS model comes with Quick Release Wheels to reduce the overall weight of your Travel Buggy
- To release the rear wheels, first unplug the quick connect cables at the back of the unit (image 1)
- Press down on the locking pin and slide it out upwards (image 2). Slightly lift the chair & pull the kick stand out (image 3)
- Slide the wheels horizontally off the locking plate (image 4)
- Reverse these steps to connect the wheels again before using your CITY 2 PLUS model

FREEWHEEL MODE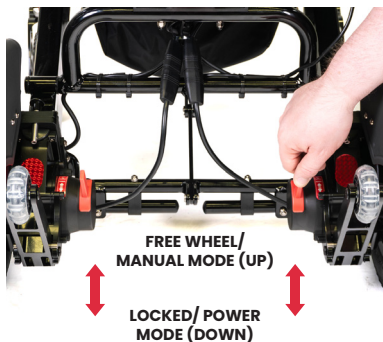


- In order to have a care giver push the CITY 2 PLUS, it must be put in Free Wheel mode
- This mode disengages the drive mechanism to the wheels, making them free to push manually
- To put the CITY 2 PLUS in Free Wheel Mode, come to a full stop and turn OFF the power on the controller.
- Push UP on the two red levers located on each rear wheel as shown

WARNING: Always make certain the wheels are properly inserted back into position and that both the teeth of the locking plate and the locking pin are in place before using your Travel Buggy.

HOW-TO FEATURES

ENGAGING ANTI-TIP WHEELS

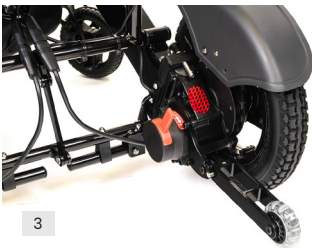- To engage the Anti-Tip wheels, pull the wheel toward you until the locking pin clears the top slot. (image 1)
- Rotate the Anti-Tip wheel down until the locking pin clicks into the bottom slot. (image 2)
- When engaged, the Anti-Tip wheels should appear like this. (image 3)

DISENGAGING ANTI-TIP WHEELS

- To disengage the Anti-Tip wheels, pull the wheel toward you until the locking pin clears the bottom slot. (image 1)
- Rotate the Anti-Tip wheels up until the locking pin clicks into the top slot. (image 2)
- When disengaged, the Anti-Tip wheels should appear like this. (image 3)

 **WARNING:** It is recommended to engage the Anti-Tip wheels before driving to ensure your safety..

 **WARNING:** It's important to disengage the anti-tip wheels before folding the power chair to ensure it stays balanced in the upright folded position.

TRANSPORTING

- It is recommended to turn OFF your CITY 2 PLUS power wheelchair. Make sure that the power chair is in freewheel mode to allow it to roll freely when pulling or loading
- Unlock the frame latch by pressing down (1) while simultaneously beginning the folding process
- Continue the folding process by holding one hand beneath the seat frame and another on the top bar (2), similar to a lawn chair
- Once folded, the power chair can balance on its rear wheels (3)
- Finally, the footrest can be used to pull the CITY 2 PLUS (5). We recommend fully extending the foot rest in order to pull more easily

1-PIECE LOAD

- To load the CITY 2 PLUS power chair into the back of a vehicle for transport we recommend utilizing the back of the vehicle as leverage
- Position the power chair against the car as indicated in Image above (Note: we recommend a blanket to reduce scratching of the vehicle)
- Grasp the rear wheels and push it into the trunk

3-PIECE LOAD

- To load the disassembled CITY 2 PLUS power chair into the back of a vehicle follow steps outlined on page 16 to remove rear wheels
- Load into vehicle as 3 separate pieces with ease

BATTERY MAINTENANCE

Lithium-Ion rechargeable batteries require routine maintenance and care in their use and handling. Read and follow the guidelines in this document to safely use Lithium-Ion batteries and achieve the maximum battery life span.

OVERVIEW

Do not leave batteries unused for extended periods of time, either in the product or in storage. When a battery has been unused for 6 months, check the charge status and charge or dispose of the battery as appropriate.

The typical estimated life of a Lithium-Ion battery is about three years or ~500 charge cycles, whichever occurs first. One charge cycle is a period of use from fully charged, to fully discharged, and fully recharged again. Use a three-year life expectancy for batteries that do not run through complete charge cycles.

Rechargeable Lithium-Ion batteries have a limited life and will gradually lose their capacity to hold a charge. This loss of capacity (aging) is irreversible. As the battery loses capacity, the length of time it will power the product (run time) decreases.

Lithium-Ion batteries continue to slowly discharge (self-discharge) when not in use or while in storage. Routinely check the battery's charge status.

Use only the provided battery(s) in your product.

BATTERY MAINTENANCE

Observe and note the run time that a new fully-charged battery provides for powering your product. Use this new battery run time as a basis to compare run times for older batteries. The run time of your battery will vary depending on the product's configuration and the applications that you run.

Routinely check the battery's charge status.

Carefully monitor batteries that are approaching the end of their estimated life.

Consider replacing the battery with a new one if you note either of the following conditions:

- The battery run time drops below about 80% of the original run time.
- The battery charge time increases significantly.

If a battery is stored or otherwise unused for an extended period, be sure to follow the storage instructions in this document. If you do not follow the instructions, and the battery has no charge remaining when you check it, consider it to be damaged. Do not attempt to recharge it or to use it. Replace it with a new battery.

CHARGING

Always follow the charging instructions provided with your product. Refer to your product's user manual and/or online help for detailed information about charging its battery.

Use only the provided charger to charge your battery(s).

BATTERY MAINTENANCE

STORAGE

- Charge or discharge the battery to approximately 50% of capacity before storage.
- Charge the battery to approximately 50% of capacity at least once every six months.
- Remove the battery and store it separately from the product.
- Store the battery at temperatures between 5 °C and 20 °C (41 °F and 68 °F).

HANDLING PRECAUTIONS

- Do not disassemble, crush, or puncture a battery.
- Do not short the external contacts on a battery.
- Do not expose a battery to fire or water.
- Do not expose a battery to temperatures above 60 °C (140 °F).
- Keep the battery away from children.
- Avoid exposing the battery to excessive shock or vibration.
- Do not use a damaged battery.
- If a battery pack has leaking fluids, do not touch any fluids.
- Dispose of a leaking battery pack (see Disposal and Recycling in this document).
- In case of eye contact with fluid, do not rub eyes. Immediately flush eyes thoroughly with water for at least 15 minutes, lifting upper and lower lids, until no evidence of the fluid remains. Seek medical attention.
- Immediately discontinue use of the battery if, while using, charging or storing

the battery, the battery emits an unusual smell, feels hot, changes color, or shape, or appears abnormal in any way. Contact Travel Buggy if any of these problems are observed

TRANSPORTATION

- Always check all applicable local, national, and international regulations before transporting a Lithium-Ion battery.
- Transporting an end-of-life, damaged, or recalled battery may, in certain cases, be specifically limited or prohibited.

DISPOSAL AND RECYCLING

- Lithium-Ion batteries are subject to disposal and recycling regulations that vary by country and region. Always check and follow your applicable regulations before disposing of any battery. Contact Rechargeable Battery Recycling Corporation (www.rbcc.org) for U.S.A. and Canada, or your local battery recycling organization.
- Many countries prohibit the disposal of waste electronic equipment in standard waste receptacles.
- Place only discharged batteries in a battery collection container. Use electrical tape or other approved covering over the battery connection points to prevent short circuits.

CHARGING

- THERE IS NO NEED TO CHARGE THE BATTERIES UPON ARRIVAL
- Your CITY 2 PLUS will arrive with fully charged batteries
- The preferred charging method (onboard charging) detailed below charges both batteries at the same time; the secondary charging method (offboard charging) described further down only charges one battery at a time

ONBOARD CHARGING (PREFERRED METHOD)

- Turn OFF the power to the CITY 2 PLUS
- Plug in the charger to the joystick controller (front of joystick) as in image 1
- This process will charge BOTH batteries simultaneously

OFFBOARD CHARGING

- Turn OFF the power to the CITY 2 PLUS
- Remove the battery(s) from the side of the chair by pulling up on the release knob as indicated in image 2
- Insert the charger end (DC) into the power socket of the battery as indicated in image 3. Second, connect the input plug of charger with your local power source.
- Connect the input
- Please ensure the charger and battery are on a flat surface and keep in a well ventilated area
- Ensure that the batteries are re-inserted correctly prior to driving

⚠ WARNING: TO REDUCE THE RISK OF INJURY, CHARGE LITHIUM-ION PROVIDED BATTERIES ONLY WITH PROVIDED LITHIUM-ION BATTERY CHARGER. Other types of chargers may cause the battery to generate heat, explode or ignite potentially causing personal injury or damage.

⚠ NOTE: When you troubleshoot battery issues for dual battery configurations, test one battery and one battery slot at a time. A defective battery can prevent the battery in the opposite slot from charging, leaving you with two uncharged batteries.

CHARGING

- Fully Charged
- Charging

**BATTERY
CHARGE
INDICATOR
ON JOYSTICK
CONTROLLER**

- When charging the indicator light on the power charger brick is red
- After 5 to 7 hours of charging (depending on the remain electric quantity, the indicator lights change from red to green, which means the battery is fully charged
- To check your battery charge, unplug the charger from the joystick and turn ON the joystick controller power
- There are a total of 7 battery power indicator lights; 3 GREEN 3 YELLOW, and 1 RED
- From top down the lights will begin to disappear as the battery(s) drain indicating the amount of charge remaining
- Recharge at any point before the last RED light disappears indicating a fully depleted battery(s)

MAINTENANCE, CARE, TROUBLESHOOTING

It is important to perform regular maintenance of your CITY 2 PLUS for longevity and warranty eligibility. Even a little cleaning now and again will go a long way to keeping your device looking and functioning like new.

Certified repair shops with trained professionals who specialize in electric wheelchairs and scooters will be qualified to perform the service and maintenance on your CITY 2 PLUS electric wheelchair should it be required. Please be sure to keep a log of the service and maintenance performed. Your shop or technician may require parts, which Travel Buggy has available.

We recommend you inspect your CITY 2 PLUS two to three times per year for in the following areas:

- **Frame:** tighten loose screws as necessary using the tool kit included with your CITY 2 PLUS.
- **Wheels:** make sure they rotate easily

Significant resistance may require new bearings

- **Armrests:** look for excessive wear or if they are loose

Replacement parts and more major maintenance issues should be done by a professional. We do not recommend that you do intense types of maintenance work to the CITY 2 PLUS yourself.

LUBRICATION

- According to use condition, please clean or lubricate the front axle, rear axle, front fork and rotating parts after 180 days
- Molybdenum disulfide grease is recommended
- Your CITY 2 PLUS power wheelchair has electric hub motors that do not require lubrication, please do not lubricate the motors

Question	Possible Causes	Answer
Why is there NO POWER to the CITY 2 PLUS?	• Controller system power is not connected. • Controller system CPU and joystick are not connected. • Battery power is too low.	• Connect the Battery. • Ensure all connections between Joystick, CPU and Battery are securely tightened. • Charge the Battery.
Why does my power chair beep and not move?	• Freewheel mode engaged • Capacity is reached • Other error code	• Ensure the rear motor switches are in POWER mode, rear wheels should be locked • Contact Travel Buggy if other error code present

LIMITED WARRANTY

The purchaser of this product is entitled to a limited warranty as offered by Travel Buggy and its affiliates on the following components and timeframes from the date of purchase:

5 YEAR WARRANTY – Main Frame

2 YEAR WARRANTY – Motor,

1 YEAR WARRANTY – Control System (CPU), Battery/Cables

90 DAY LIMITED WARRANTY – On the wearing parts, such as the tires, armrest padding, joystick button covers, controls knobs, seat cushion, and backrest padding.

This warranty is valid for the replacement of dysfunctional parts only. Any parts under warranty will be replaced and shipped to your door. Any service and labor fees, if applicable, to replace parts under warranty must be paid by the user.

Due to its straightforward design, most parts can be easily exchanged by the end user without a professional service tech required. However, it is always recommended you seek professional help for maintenance and service, to make sure the work is done properly.

The warranty only covers NON-DELIBERATE Damage.

The warranty does not cover:

- Products damaged by user negligence.
- Products damaged accidentally.
- Products damaged intentionally.
- Products that have been subjected to negligence.
- Products that have been subjected to abuse.
- Products that have been improperly stored.
- Products that have been improperly handled.
- Products that have been improperly operated.
- Products that have experienced general misuse.
- Products that have been modified in an unauthorized, unapproved way.

Warranty is non-transferable and only valid for the original wheelchair purchaser.

Travel Buggy will, at its sole discretion, replace any warrantable item that fails within the warranty period.

SERIAL NUMBER

The serial number on the CITY 2 PLUS is located on the frame under the seat. It is important to mention the serial number in any warranty related communications

CONTACT & SUPPORT

You will find the operation and care of your CITY 2 PLUS very simple and easy, but should you have any questions please contact Travel Buggy.

OUR CUSTOMER HOTLINE

Canada: 1.888.56.BUGGY (28449)

USA: 1.855.56.BUGGY (28449)

EMAIL

contact@travelbuggy.com

Additional images and videos are available on our website.

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