

# HOVEROUND

## Stairlifts



# H1100 Straight Stairlift

With You Every Step of the Way



“My stairlift gives me complete **independence** and fills me with total **confidence** in moving around my home safely every day.”

### Features

- Patented Multi-Drive Technology
- Ultra Slim Rail
- Constant Charge
- Compact and Foldable
- Easy to Use and Maintain
- Available Power Swivel and Powered Footrest Options



### Safe Working Load (SWL)

30–45°: 309 lb (140 kg)  
46–50°: 287 lb (130 kg)



### Warranty

Limited Lifetime Warranty on Motors and Gearbox  
2 Years on Materials and Workmanship

## Regain your independence and conquer the struggles of staircases.

Hoveround offers stairlift solutions that provide freedom of movement throughout your home, allowing you to age in place free from everyday obstacles. Our compact and customized stairlifts can be installed nearly anywhere. Let us help see if the all new H1100 stairlift is right for you!

## Ultra slim profile. Revolutionary drive system.


The H1100 Straight Stairlift features next generation technology with an ultra-streamlined design that takes up less space on your stairs. The Patented Multi-Drive Friction System will ensure your stairlift is operating at peak levels at all times with no reduction in performance ever.

The H1100 has eliminated the traditional greasy rack and pinion model and instead utilizes a state-of-the-art friction drive system to power this next gen stairlift. We've achieved a quieter, smoother, more robust stairlift that's much easier to maintain. The contemporary design allows the H1100 to blend into any environment, becoming a complementary piece in your home.



### Did You Know?

The cost of a stairlift is normally only a fraction of the cost of leaving the home you love.

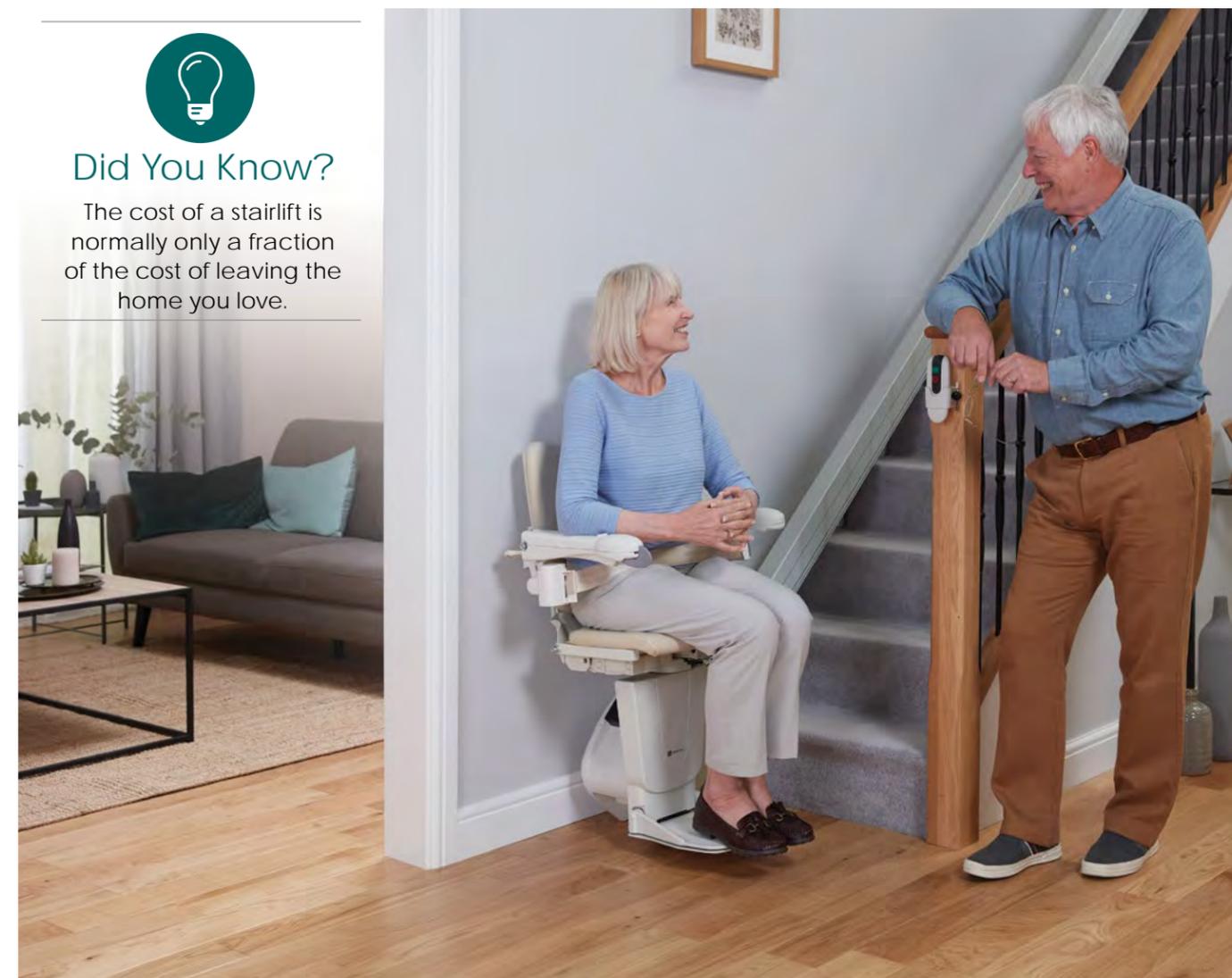

# Features for your **Safety** and **Convenience**



## Compact

The H1100 folds up easily maximizing the space for family members to go up and down the stairs as needed.

We know back pain and aches can occur anytime, that's why the H1100 was made to be easy to upgrade. Powered footrest and seat swivel can be added after installation for added safety and convenience.



## Easy to Use

Ergonomic and intuitive to use, the joystick on the armrest smoothly operates the H1100.

Send for your stairlift at the touch of a button with the included remote control. Two are included with your H1100 with holders for both the top and bottom landings.



## Easy to Maintain

Pet owners rejoice, the H1100 features four brush heads that constantly clean the rail from dust and hair, ensuring your lift is always spotless. The tooth rack free design also eliminates the necessity for grease on the rail offering a clean finish today and tomorrow.



## Always Charged

The H1100 comes standard with Continuous Charge, your lift is always being charged anywhere on the track!



## Did You Know?

The H1100 technology ensures an exceptionally smooth ride up and down the stairs; when you reach the top landing or bottom the H1100 gently glides to a halt, avoiding an abrupt sudden stop.



## Retractable Seatbelt

The H1100 comes equipped with an easy-to-use retractable seat belt, a standard feature that provides added security when safely seated in your stairlift.



## Stairlift Myth

"Maintaining a stairlift is expensive." Cost-effective parts, maintenance friendly design, and Hoveround's industry-leading warranty make the H1100 the smart choice.



## Patented Multi-Drive Friction Technology

The first and only multi-drive system in a stairlift today, the H1100 uses 4 independent motors to provide a smoother, safer, more comfortable ride. In the unlikely event of a motor issue, you have three others to transport you safely to the top or bottom of the stairs. An emergency stop button is also located on the bottom of the chair.



## Safety First

The optional powered footrest allows you to fold and unfold the footrest without having to bend down, an anti-slip texture on the footrest provides a better foot grip when seated and departing.

The H1100 is also fitted with multiple safety sensors which stop the stairlift whenever it runs into an obstacle.



## Swivel Automatically

The optional powered swivel makes it easy to get off the lift at the top of landings, simply hold the toggle switch in the direction of the landing to automatically turn the seat for a controlled departure.

With 80 degrees of tilt, you face more of your top landing providing a greater security blanket when transferring to and from your H1100.



### H1100 Fact!

The H1100 can be installed quickly – our Home Access Specialists are trained to be efficient; stairlift installation should be quick and pain-free and shouldn't interfere with your day.



Vanilla Beige

## Frequently Asked Questions

We strive for 100% customer satisfaction. That is why we go the extra mile to offer you the highest standards of service and answer your questions in the best way possible. We partner with the industry's leading manufacturers that have a track record of the highest quality and best value equipment on the market.

We have listed some of our customers' most frequently asked questions. Should you have further questions, please contact a friendly Hoveround mobility specialist today!



### Can a stairlift be installed on my staircase?

Hoveround offers a wide selection of stairlifts for straight and curved stairs. With a variety of lift models we are able to provide a suitable solution for each setting regardless of whether the stairs are narrow, steep, curved or long, indoor or outdoor. Hoveround has a solution for nearly any staircase.

### Does it take major renovation to install a stairlift in my home?

No! The H1100 Straight Stairlift is simply installed onto the stair treads. Because it is fixed to the staircase, you don't have to worry about any further modification to your home.

### What happens to my stairlift during a power outage?

A power failure is no reason to worry. Your Hoveround stairlift will use its batteries to remain operational and can still be used for at least 10 rides up and down the stairs. The H1100 features Constant Charge, providing charging anywhere on the track so it is ready for use 100% of the time.

### How do I know when batteries need to be replaced?

The H1100 is equipped with a fault display behind the power pack, simply review the code and call Hoveround and we can easily look up the code and inform you if the battery needs to be replaced.

Generally the batteries will last for three years depending on usage.

### How do I enter into and exit out of a stairlift?

Entering and exiting the H1100 is made easy, simply unfold the seat and footrest, sit down while securing the seat belt. When it's time to exit your H1100 you can use easily swivel the seat to face your landing for a controlled departure.

### Can others still use my stairs?

Compact and ultra slim, the H1100 can easily be folded up when not in use to allow for easy pass-through for family members and friends.


## A Stairlift Suited for Any Interior

The H1100 was designed to be a smart, stylish complement to everything else in your home. The contemporary design is welcome in any home décor and is available in a warm vanilla beige upholstery.

Featuring our comfortable Style Seat in a leatherette finish, the seat surface is easily wipeable and cleans effortlessly.

The ultra slim rail beautifies the H1100, simply there is no other rail on the market that competes with its overall look and sleek profile, a non-obtrusive design that quietly blends in with your staircase.

# H1100 Straight Stairlift Specifications

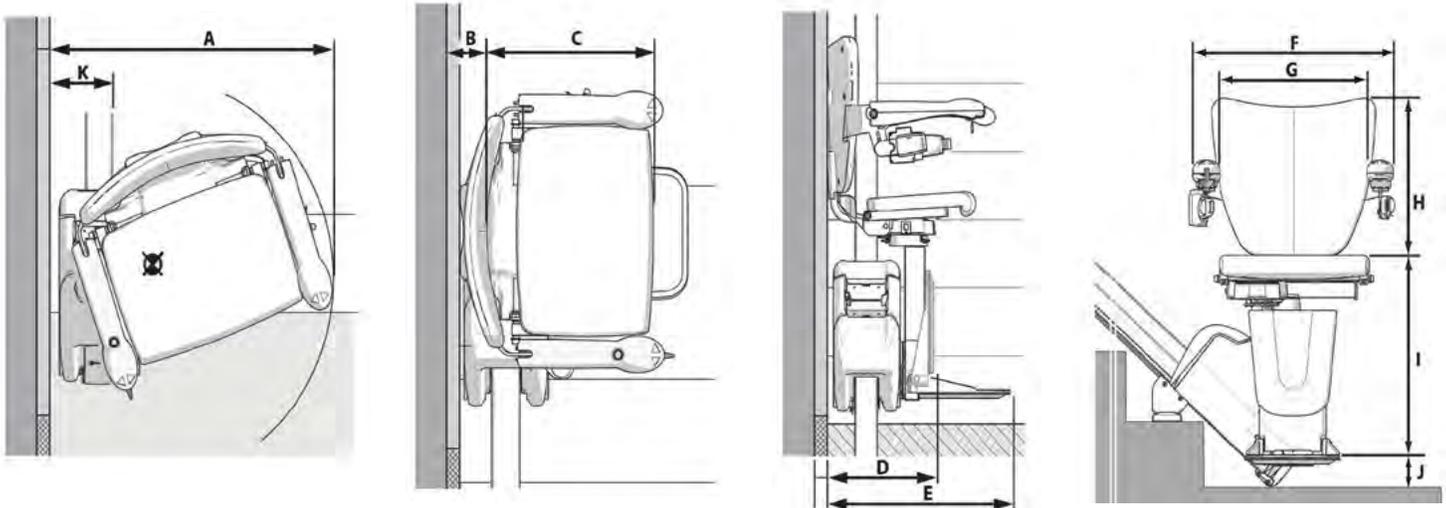

## H1100 Style Seat Dimensions

### Safe Working Load (SWL)

30–45°: 309 lb (140 kg)

46–50°: 287 lb (130 kg)

### Staircase Angle

30–50°

### Speed

0.5 ft / sec (0.15 m / sec)

Some of the dimensions depend on the characteristics of the staircase in the home. Manufacturer reserves the right to alter designs and specifications without prior notice.

A	Swivel radius, minimum	25.4 in (645 mm)
B	Wall to inner seat back	2.4 in (60 mm)
C	Seat depth	15.4 in (390 mm)
D	Folded width to footplate, minimum	12.6 in (320 mm)
E	Open width to edge footplate, minimum	20.9 in (530 mm)
F	Armrest width, external	22.6 in (575 mm)
G	Armrest width, internal	17.7 in (450 mm)
H	Seat back height	15.7 in (400 mm)
I	Footplate to seat height	16.9–20.5 in (430–520 mm)
J	Footplate height, minimum	2.75 in (70 mm)
K	Track intrusion into staircase, minimum	6.3 in (160 mm)

Are you ready to take back your independence with the H1100?

Call 1-800-625-2190 or visit [www.hoveround.com/stairlifts](http://www.hoveround.com/stairlifts)

**HOVEROUND**  
Stairlifts

Hoveround Corporation  
2151 Whitfield Industrial Way, Sarasota, FL 34243  
Toll Free: 800-000-0000 | Fax: 800-337-0367  
[stairlifts@hoveround.com](mailto:stairlifts@hoveround.com) | [www.hoveround.com](http://www.hoveround.com)