



## HOW TO INSTALL YOUR COLOR TRIM KIT

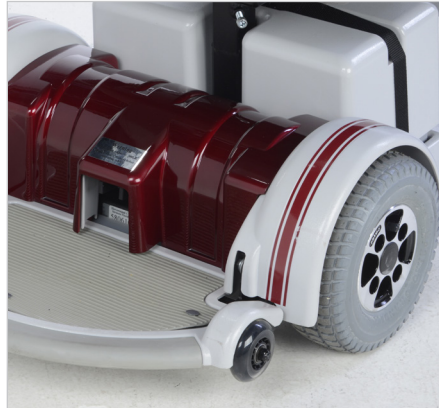

### WHEEL HOUSING DECAL

Remove existing stripe. Peel backing from new stripe. Beginning from front side. Carefully place new strip in recessed area. Repeat on other side.

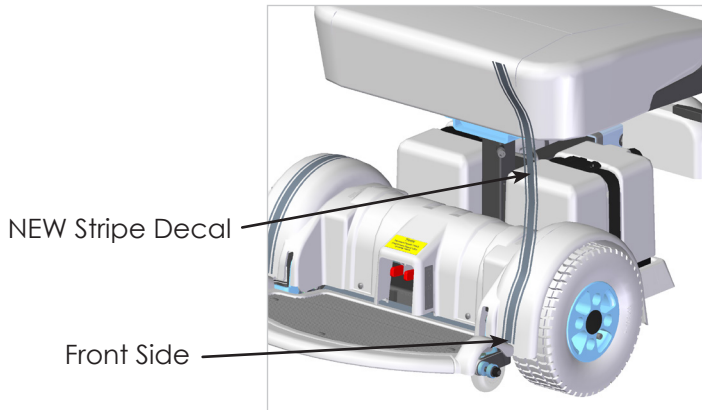

### MOTOR COVER

Align new motor cover trim directly over top of existing motor cover and press down firmly until there are no gaps between the new cover and the body plastic.



### OUTRIGGER COVER

Remove backing from foam tape on underside of new outrigger cover. Then align directly over top of existing outrigger plastic and press down firmly until there are no gaps between the new cover and the outrigger plastic.

### ⚠ WARNING

To avoid serious injury or death:

- **ALWAYS** turn the power off before and during repair operations.
- **BE SURE** all plastic moldings are secure before driving.